Freezer Temperature Log

Month	Noarl	Dave	1_15)
women	/ Tear i	Days	1-12

Freezer Location _____

Record CURRENT, MIN, and MAX temperatures twice a day. Complete steps 1-3

Step 1

Write your initials and time of day.

Provider Name:	
Pin number:	

File this log at the end of the month and keep it for 3 years

Staff Initials	Example	2																													
Day of Month	1	1	L	2	2		3	4	ŀ	- 2	5	Ű	6	•	7	8	8	U.	9	1	0	1	1	1	2	1	.3	1	.4	1	15
	7:30																														
Time	am	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm
Current	-22																														
MIN	-25																														
MAX	-19																														



Step 3

Read the current temperature on thermometer display and record on the temp log. Review the minimum and maximum temperatures and record on the temp log.

Evaluate temperatures. If temperatures are in the DANGER zone (see zone below),



Review/Mark Button: Press to see min/max temperatures

Start/Clear/Stop Button: Press to return to normal mode

-25° & lower -24° -23° -22° -21° -20° -19° -18° -17° -16° -15°	-14° -13° -12° -11° -10° -9° -8° -7° -6° -5° -4° & higher
THESE TEMPERATURES ARE OK	DANGER ZONE - TOO HOT
 Vaccine Storage Best Practices Store vaccine at -18° or lower to reduce chance of temperatures exceeding the recommended range Place thermometer in a central area of the unit directly with the vaccines Do not place thermometer or vaccine in the door, near or against the walls, close to vents, or on the floor of the unit 	 If you ever see temps in this danger zone: Alert your supervisor immediately Wait no more than 60 min and check to see if the temperature has returned to the proper range If temps are still in DANGER ZONE after 60 minutes, download the data and email to chicagovfc@cityofchicago.org Document the date and actions you take on the Vaccine Storage Troubleshooting Record

take immediate action described below.

~ 0	

Freezer Temperature Log

Pin number:_____

Freezer Location _____

Provider Name:_____

Step 1 Write

Write your initials and time of day.

onep 1		File this log at the end of the month and keep it for 3 years																															
Staff	Example RC																																
Day of Month	15	1	6	1	.7	1	8	1	.9	2	20	2	21	2	22	2	3	2	4	2	5	2	6	2	7	2	8	2	9	3	80	3	1
Time	7:30	fan en en en																															
	am	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm	am	pm
Current	-22																																
MIN	-25																																
MAX	-19																																



Read the current temperature on thermometer display and record on the temp log. Review the minimum and maximum temperatures and record on the temp log.

Record CURRENT, MIN, and MAX temperatures twice a day. Complete steps 1-3

Step 3 Evaluate temperatures. If temperatures are in the DANGER zone (see zone below), take immediate action described below.



Review/Mark Button: Press to see min/max temperatures

Start/Clear/Stop Button: Press to return to normal mode

-25° & lower -24° -23° -22° -21° -20° -19° -18° -17° -16° -15°	-14° -13° -12° -11° -10° -9° -8° -7° -6° -5° -4° & higher
THESE TEMPERATURES ARE OK	DANGER ZONE - TOO HOT
 Vaccine Storage Best Practices Store vaccine at -18° or lower to reduce chance of temperatures exceeding the recommended range Place thermometer in a central area of the unit directly with the vaccines Do not place thermometer or vaccine in the door, near or against the walls, close to vents, or on the floor of the unit 	 If you ever see temps in this danger zone: Alert your supervisor immediately Wait no more than 60 min and check to see if the temperature has returned to the proper range If temps are still in DANGER ZONE after 60 minutes, download the data and email to chicagovfc@cityofchicago.org Document date and actions you take of the Vaccine Storage Troubleshooting Record